

Pre-filled pen for multiple injections
GONAL-f[®] Pen

Four strengths
150 IU, 300 IU, 450 IU and 900 IU



How to use your pre-filled GONAL-f pen for multiple injections


Three simple steps



1

Attaching the needle


- Remove the pen cap.
- Check that the dose window shows "0".
- Remove the peel-off seal of the outer needle cap.
- Press the threaded tip of the pen into the outer needle cap and twist it clockwise until you feel a light resistance.
- Do not attach the needle too tightly, as it may be difficult to remove it after injection.

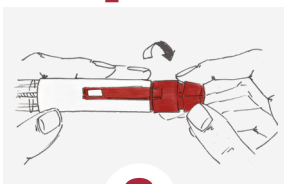

2

Setting the dose

- Turn the dose setting knob forward until your prescribed dose shows in the dose window. You can adjust the dose in increments of 12.5 IU.
- If necessary, correct the dose by turning the knob backwards.
- Always check before injecting, that the dose window shows your prescribed dose.

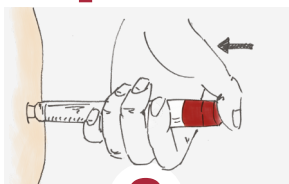

3

Injecting the dose

- Push the needle slowly into the skin at a 90-degree angle.
- Press the injection button down as far as it will go. The larger your dose, the longer it will take to inject. Keep the needle in the skin with the injection button pressed down for at least 5 seconds. The dose window should now show "0".
- Keep the injection button pressed down while removing the needle from your skin.
- Vary injection sites to minimize skin irritation.

Things to be considered

Always dispose of the needles as instructed.

If you need more information, ask your doctor or pharmacist.

DO NOT REUSE THE NEEDLE.

Write down the first day of pen use.

Once you have opened the package, the pre-filled pen may be stored outside the fridge for a maximum of 28 days (at a temperature of 25 °C).

Do not use the product after the expiry date ("EXP"). The expiry date refers to the last day of that month. Do not use the product, if there are particles in your medicine or the medicine looks cloudy.

Storage:

FRIDGE

4 °C



25 °C
15 °C



hedelmöityshoito.fi



Use this QR
for login.

More information and an instructional video are available on the website.

Log in to the site using the six-digit Nordic VNR number found on the carton of your medicine.



FI-GON-00040