

OVITRELLE[®]

Choriogonadotropin alfa



Ovitrelle pre-filled pen
250 mcg/0.5 ml

How to use your OVITRELLE pre-filled pen

Three simple steps




1

Preparation

Read the instructions for use and the package leaflet carefully before each injection


- Remove the peel-off seal.
- Screw the threaded tip of the Ovitrelle pre-filled pen into the outer needle cap until you feel a light resistance. Do not attach the needle too tightly – the needle could be difficult to remove after the injection.
- Remove the outer needle cap.

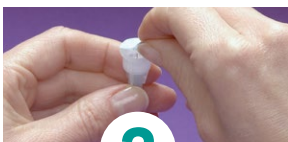

2

Setting the dose

- To set the dose, turn the dose setting knob clockwise until the number 250 is shown in the dose display. Do not push or pull the dose setting knob while you turn it.

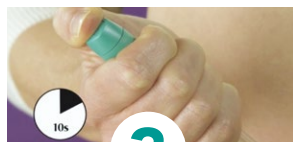

3

Injecting the dose

- Choose the injection site according to the instructions given by your doctor or nurse.
- If you wish, you can clean the injection site with an alcohol swab.
- Verify that the number in the dose display reads '250'. Carefully remove the inner needle cap.
- Slowly push the needle into the skin entirely and then press the dose setting knob as far as it will go. Keep the needle under the skin with the dose setting knob held down for a minimum of 5 seconds.
- Keeping the dose setting knob pressed down, pull the needle out of the skin. Then release the dose setting knob.
- Verify that the number in the dose display reads '0'. This shows that the complete dose was delivered.

After the injection

- Carefully put the outer needle cap onto the needle.
- Grip the outer needle cap and unscrew the needle.
- Put the pen cap back onto the pen.



Choriogonadotropin alfa



Ovitrelle pre-filled pen
250 mcg/0.5 ml

Safety

Always dispose of the needles as instructed.

If you need more information, ask your doctor or pharmacist.

NEVER REUSE A NEEDLE THAT HAS ALREADY BEEN USED

How to store Ovitrelle:

REFRIGERATOR

2–8°C Do not
freeze.

Take the pen out of the refrigerator approximately
30 minutes before the injection.

This will increase your comfort during the injection.

Do not use the product after the expiry date ("EXP").

The expiry date refers to the last day of that month.

Do not use the product if you notice any particles in the
medicine or the medicine looks cloudy.

Merck Oy
Keilaranta 6, FI-02150 Espoo
www.merck.fi

MERCK